


plus One[®]

prostate massager




instruction manual

product overview

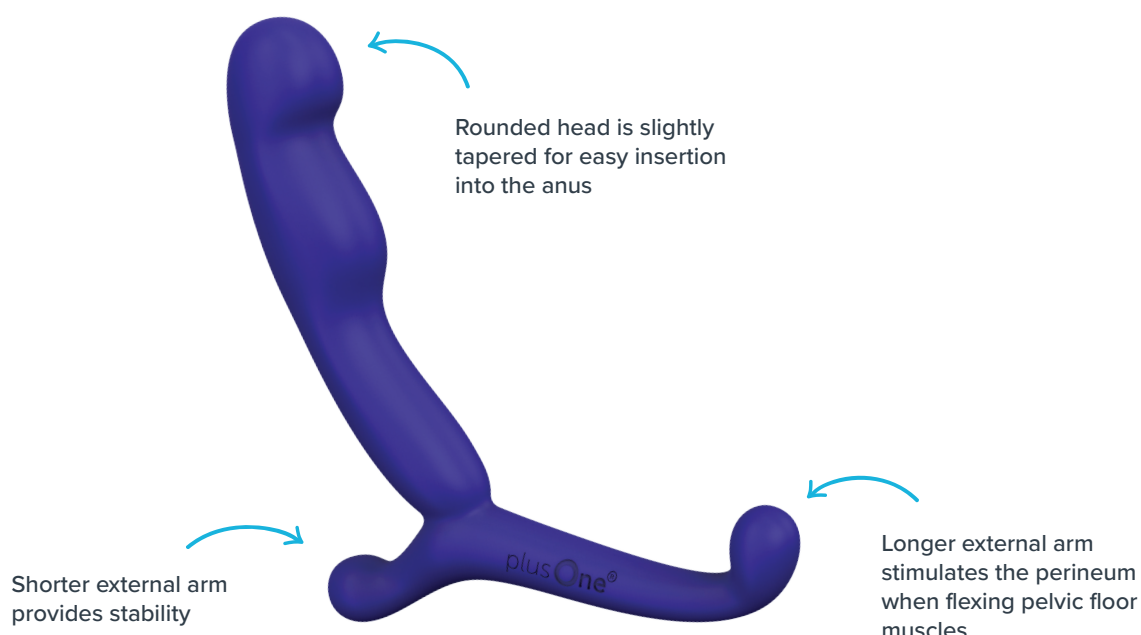

prostate health

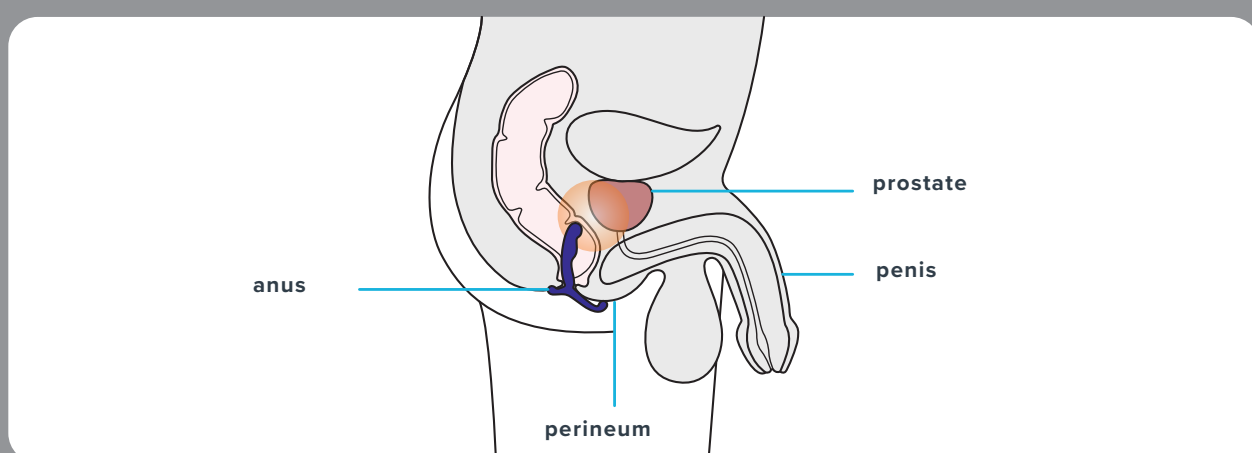

Stimulating the prostate helps release inflammation, pressure, and increases blood flow in the pelvic area. When flexing pelvic floor muscles, the prostate massager rocks back and forth applying direct pressure and intense stimulation to the prostate.

special instructions:

Always apply adequate, water-based lubricant to the massager when inserting. Do not force insertion.

cleaning:

Clean thoroughly with mild soap and warm water, before and after each use.

storage:

Store in a cool dry place separately from other sexual wellness devices; sexual wellness devices of different material can damage each other if left in contact.

quality:

This product is manufactured using body-safe materials that is harmless to the human body; free from phthalates, BPA and latex.


warnings:

Discontinue use if there is any discomfort for you or your partner.


ways to use



Apply a generous amount of hydrolube™ personal lubricant to the rounded head for easy insertion into the anus. Use with water-based lubricant only.

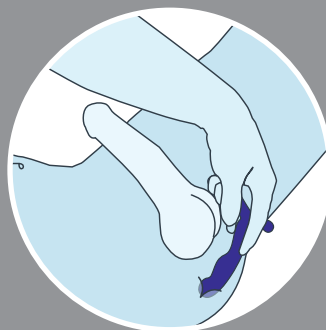

Gently insert the head of the prostate massager into your anus, positioning the massager's perineum arm against the area between the anus and the scrotum. Contract your sphincter muscles several times to pull the prostate massager in, applying pressure to the prostate. Do not force insertion. Allow your body to become accustomed to the massager.

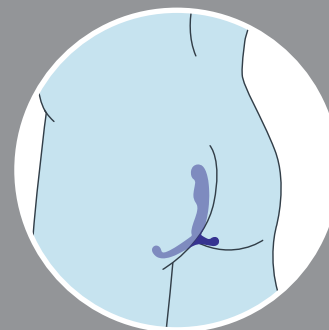

Contract and relax the sphincter muscle as slowly as possible. Your prostate massager will automatically massage the male g-spot. You can also use your hands to manually apply pressure to the area.

The battery shall be disposed in accordance with the local laws and regulations. If possible please recycle it. Users shall not take out or replace the battery by themselves; doing so may damage the product.

two year limited warranty:

Your new plusOne® products carry a warranty against manufacturing defects for two years from the date of original purchase. plusOne® will, at our option and upon submission of a valid proof of purchase within the warranty period, repair or replace the product free of charge.

For assistance or for issues related to performance, damages, warranty, regular maintenance or customer feedback please contact plusOne® toll free at (800) 514-2546, Monday – Friday from 8:30 am until 5:30pm EST or email us at info@myplusOne.com